GR
EN

REF: 0806421

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T: +30 28210 63222 | WEB: www.mobiakcare.com

MOBIAC®
MEDICAL • GASES • FIREFIGHTING est. 1977

Intended Use:

The examination bed is intended for use in a Medical Office, in a Physiotherapy Clinic or in any other place for the diagnosis of patients and the short-term application of physiotherapy.

Indications:

Examination beds are essential in clinics, hospitals, and healthcare settings, providing a stable and supportive surface for patients during routine medical examinations, diagnostic procedures, and minor treatments. They are ideal for general assessments, including checkups, auscultations (listening to heart and lungs), ensuring patient comfort and safety during various medical evaluations.

Contraindications:

Examination beds are not be suitable for patients with severe mobility issues. For patients with specific conditions, such as pressure sores, injuries, or sensitivities (require specially designed beds). For patients requiring intensive monitoring or life support.

General Warnings:

- If you don't read this user manual, it is preferable not to use this product or another available part.
If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the device from heat sources.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.

The Package Includes:

1. The Examination Bed
2. Cushion for the backrest hole.
(Concerns items produced from September 2021 onwards)
3. Paper Mechanism
4. Hand control
5. Foot Pedal

Before use:

1. Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame).
2. Check, if the product is properly assembled.
3. Pay attention to the locking of the knobs & screws.
4. Make sure that the floor surface that will support product is flat.
5. Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects.

During The Use:

Avoid contact of the product with creams and massage oils, always placing a bed cover or other protection between the bed mattress and the patient.

After each patient you should change this cover or other protection that you have used and before the next session put a new one.

Maintenance & Cleaning:

- Clean the product surface after each use with a soft cloth.
- Dirt, oil or debris must be removed from the product surface.
- Use only a cloth and mild soap on surfaces that need cleaning.
- Then use a clean cloth and dry well before using the product. Do not leave the product wet.
- If you need to disinfect the product, use a common, mild disinfectant.
- Protect the item from scratches, cuts and punctures.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual.
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice.
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.

Environmental Protection:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.
- 4) The electrical products due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards.

Declaration of Conformity:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

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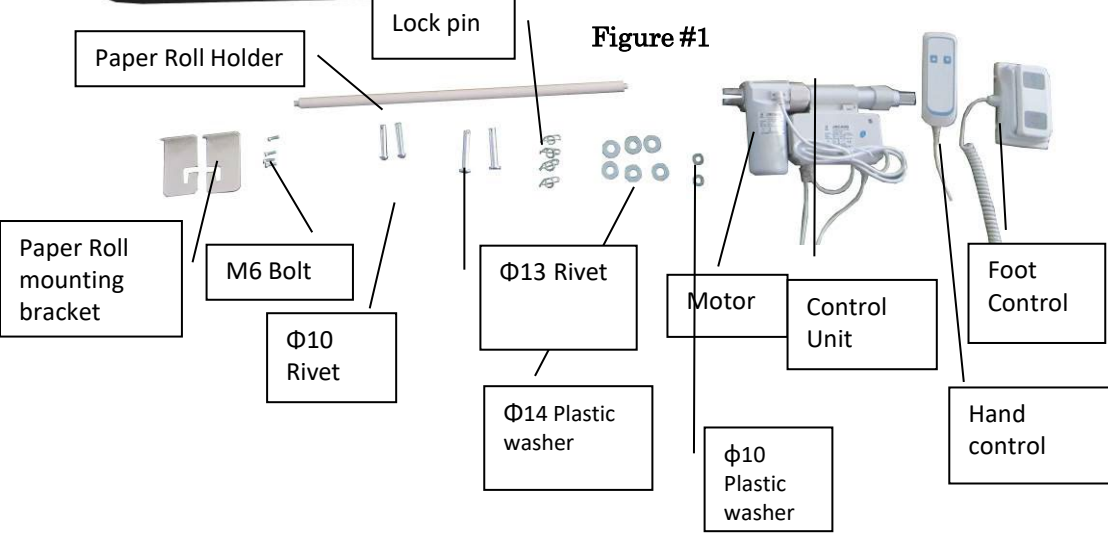
1. Installing the lifting frame: Raise the lifting frame, matching with the holes, put through the $\phi 13$ rivet with $\phi 14$ plastic washer and lock pin as Figure #2.

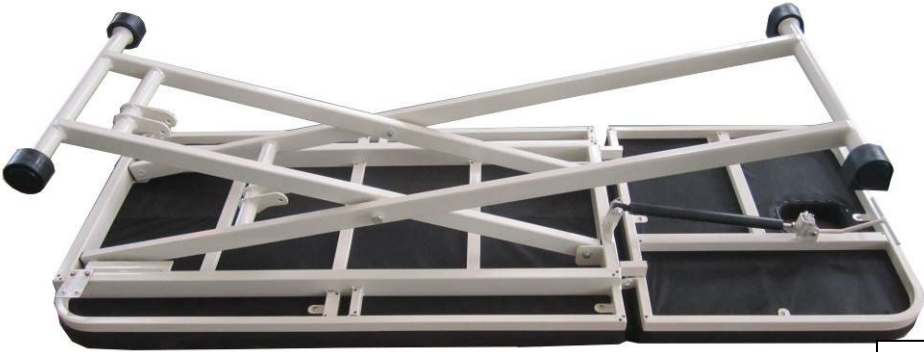
Figure #2

Lock Pin

$\Phi 14$
Plastic
washer

$\Phi 10$ Plastic washer

Lock Pin

2. Installing the motor:

Matching the motor installation hole with motor base mounting hole, put through the $\phi 10$ rivet with $\phi 10$ plastic washer and lock pin as Figure #3.

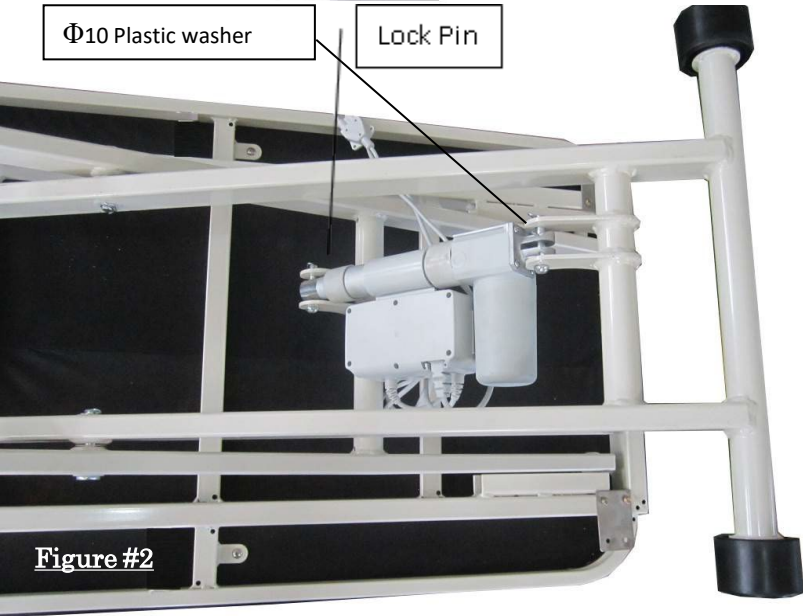
Figure #2

3. Installing the paper roll holder

3.1 Matching the hole on the holder mounting bracket with the bolt hole on the bed frame, tighten with M6 bolt as Figure #4.

3.2 Reverse the whole bed and install the paper roll holder as Figure #5.


4. Operation for hand control and foot control

First ensure the power cord is connected, press the button of hand control or step on the foot control to adjust the entire bed height, and then release the control button until the desired position. See Figure #6.

Hand Control

Figure #6

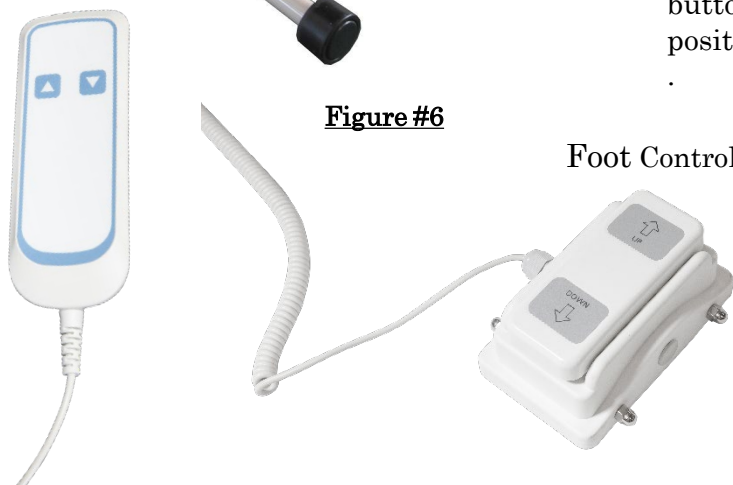Foot Control

5. Operation for backrest tilting with the gas strut

To raise the backrest: Pull up the handle with one hand and with the other hand, lift back section up to the desired angle. Release the handle and the back section will lock into place.

To lower the backrest: Pull up the handle with one hand and with the other hand, push the back section down to the desired angle. The backrest will lock into place. See Figure #7.

Figure #7

Warranty:

The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty covers factory defects & **DOES NOT** cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also **DOES NOT** cover the parts that can wear out during use, like the mattress upholstery or over time.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty.

Warranty Claim Form:

Buyers Data

Name:

Date of purchase:S/N.....

LOT:.....Phone:

Resellers Data

Name :

Address:.....

Phone:

Reseller Stamp

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

MOBIAC S.A.
ΚΑΘΙΑΝΑ ΑΚΡΟΤΗΡΙΟΥ, ΧΑΝΙΑ - ΚΡΗΤΗ
KATHIANA AKROTIRIOU CHANIA - CRETE
T: +30 28210 63222 | WEB: www.mobiakcare.com

MOBIAK®
MEDICAL • GASES • FIREFIGHTING est. 1977

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Description of symbols that have been printed on the label, Master Carton and/or User Manual

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