REF: 0808878

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T: +30 28210 63222 | WEB: www.mobiakcare.com

Intended Use:

The examination bed is intended for use in a Medical Office, in a Physiotherapy Clinic or in any other place for the diagnosis of patients and the short-term application of physiotherapy.

General Warnings:

- If you don't read this user manual, it is preferable not to use this product or another available part.
If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the device from heat sources.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.

Before use:

1. Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame)
2. Check, if the product is properly assembled.
3. Pay attention to the locking of the knobs & screws
4. Make sure that the floor surface that will support product is flat.
5. Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects.

During The Use:

Avoid contact of the product with creams and massage oils, always placing a bed cover or other protection between the bed mattress and the patient.

After each patient you should change this cover or other protection that you have used and before the next session put a new one.

Maintenance & Cleaning:

- Clean the product surface after each use with a soft cloth.
- Dirt, oil or debris must be removed from the product surface.
- Use only a cloth and mild soap on surfaces that need cleaning.
- Then use a clean cloth and dry well before using the product.
Do not leave the product wet.
- If you need to disinfect the product, use a common, mild disinfectant

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.

Environmental Protection:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

Declaration of Conformity:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

Assembly Instructions

Examination bed installation parts listed below as Figure #1:

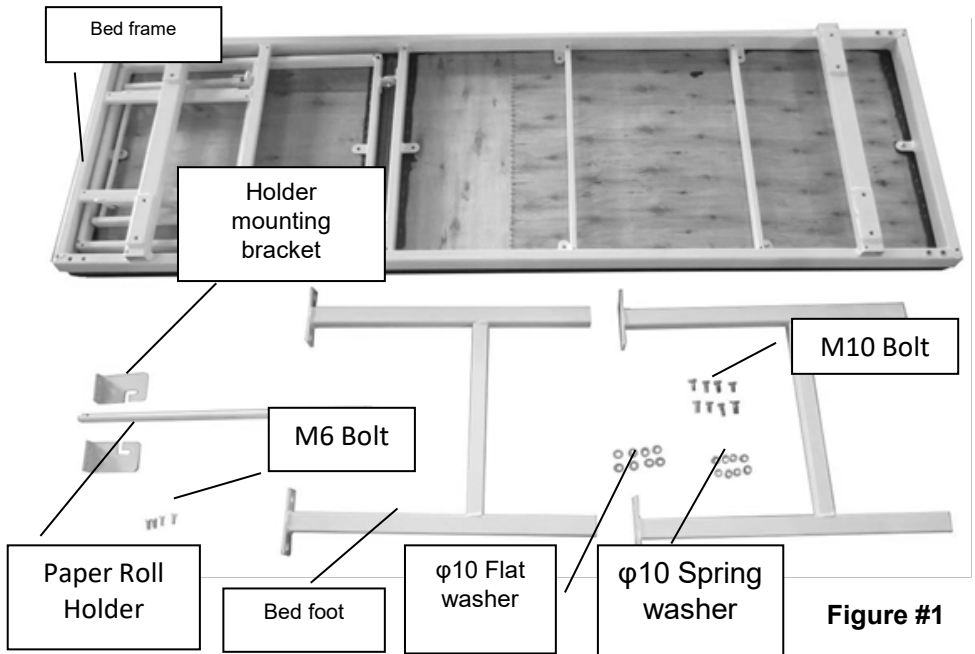
Figure #1

1. Installing the bed foot, Matching the bolt hole of the bed foot with the hole on bed frame, tighten with the bolt as Figure #2.

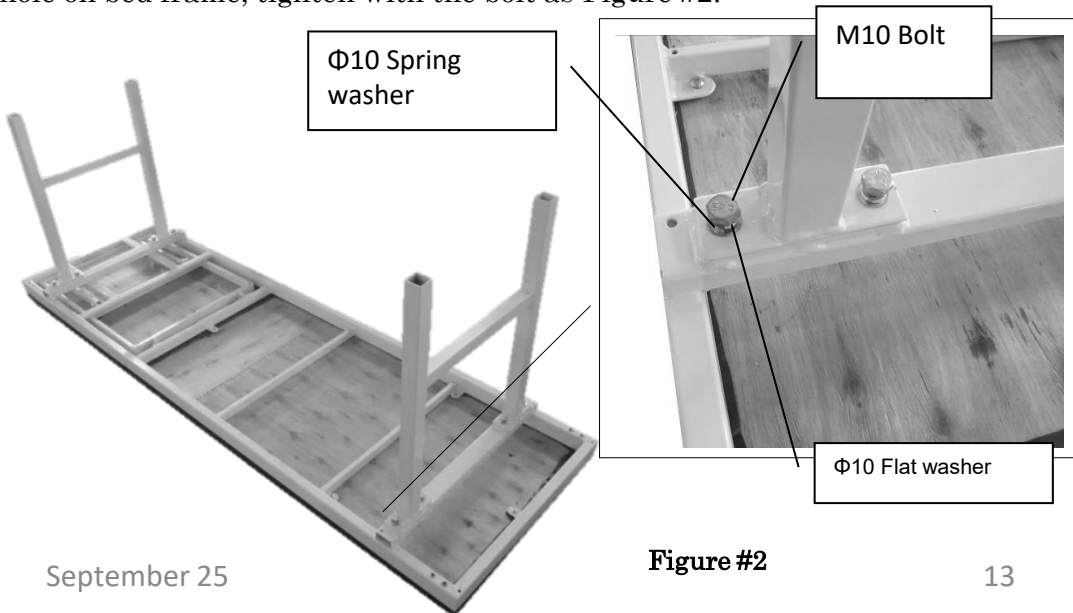
Figure #2

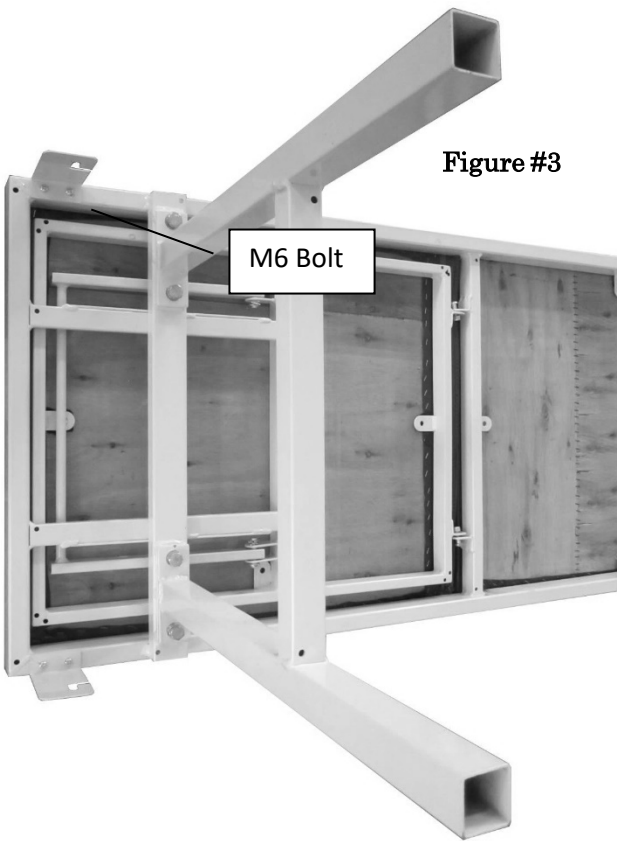
Figure #3

2. Installing the paper roll holder
 2.1 Matching the hole on the holder mounting bracket with the bolt hole on bed frame, tighten with M6 bolt as Figure #3

2.2 Reverse the whole bed and install the paper roll holder as Figure #4.

Figure #4.

3. Operation for back section

Back-rest tilting is adjustable, there are 4 slots on the bed frame for tilting adjustment at 4 degrees. The support stand is not in the slot when the back section is flat. When adjust the back-rest tilting, hold up the back section and insert the support stand into the slot at ideal degree. See Figure #5

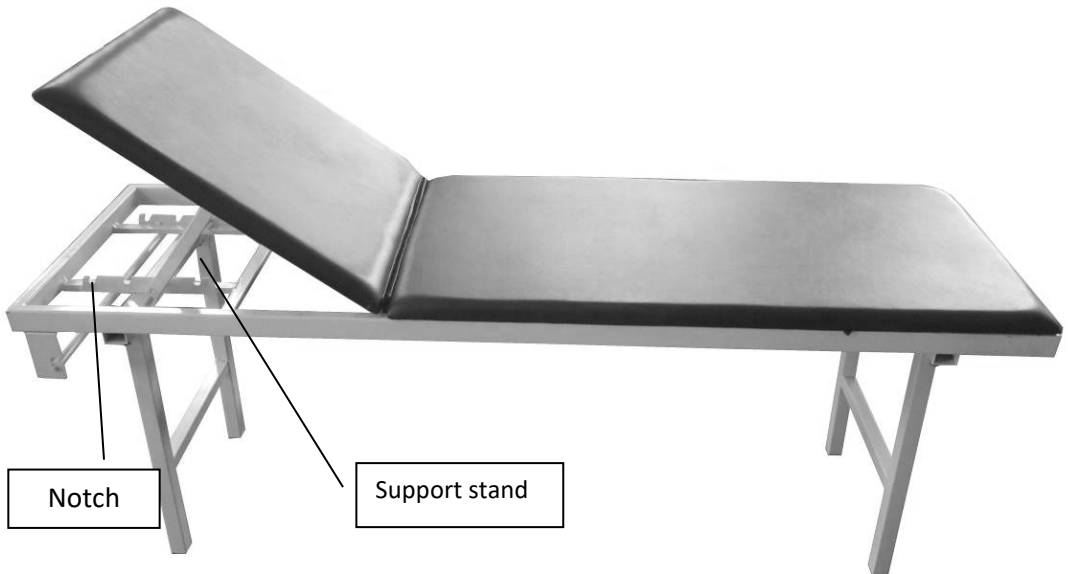
Figure #5

Warranty:

The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty covers factory defects & **DOES NOT** cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also **DOES NOT** cover the parts that can wear out during use, like the mattress upholstery or over time.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty.

Warranty Claim Form:

Buyers Data
Name:
Date of purchase:S/N.....
LOT:.....Phone:

Resellers Data
Name :.....
Address:.....
Phone:

Reseller Stamp

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

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