
TAVOLO PER MASSAGGI PORTATILE A 2 SEZIONI
2 SECTION PORTABLE MASSAGE TABLE
TABLE DE MASSAGE PORTABLE EN 2 SECTIONS
2-TEILIGER TRAGBARER MASSAGETISCH
CAMILLA DE MASAJE PORTÁTIL DE DOS SECCIONES
MESA DE MASSAGEM PORTÁTIL DE 2 SEÇÕES

GIMA 44020, 44021, 44024

Fabbricante/Manufacturer/Fabricant/Fabricante/
Hersteller:
Anji Sukar Furniture Co., Ltd.
Tangpu Industrial Zone, Dipu Town, Anji,
Zhejiang Province, China. P.C.: 313300
peter@zjruixiang.com

REF AT243

Importato da/imported by/importé par/eingeführt von/
importado por:

Gima S.p.A.

Via Marconi, 1, 20060 - Gessate (MI) - Italy
gima@gimaitaly.com - export

gimaitaly.com **www.gimaitaly.com**

Made in China



PRECAUTIONARY INSTRUCTIONS

1. Carry out periodic maintenance on the table. Make sure all screws are secure and the cable is not deteriorated or frayed. Make sure there are no cracks in the aluminum structure.
2. Do not overload the table beyond the recommended weight limit. Doing so could result in serious injury to the user. Maximum working load of 180 kgs.
3. Do not use alcohol-based cleaners when cleaning the PU LEATHER. They could cause damage to the PU LEATHER material. Use warm soapy water to clean, then wipe dry.
4. When using the table, make sure that all four legs are at the same level. Using the legs at different levels can cause the table to be unstable and could lead to injury.
5. Do not allow children to assemble or disassemble the massage table, unless under your supervision.
6. Be careful when using sharp objects, as they can pierce the covering. Keep the table's covering away from sharp items.



SETTING UP YOUR TABLE

Steps are as below:



1




2



3



4

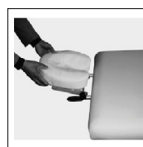

5



6

1. Release the latch.
2. Lay the table on its side with four rubber feet on the floor, open up the table with four legs extended as fully as possible.
3. Take out all accessories and then lift the table up until all legs are on the floor.
4. Put the headrest cradle into the corresponding lodging, fasten the headrest cushion on the cradle.
5. Insert the two side arm supports into the two holes at the side of the table.
6. Now the table is ready for your use.

The height of table is adjustable by pressing the copper buttons on the leg. All four legs should be adjusted in the same way, one by one, being sure to adjust all legs at the same height level.

CLOSING AND TRANSPORTING THE TABLE

1. Take out all accessories.
2. Lay the table on its side with four rubber feet on the ground.
3. Put all accessories under the elastic strap beneath the top veneer.
4. Fold the table up and lock the latch.
5. Put the massage table into the carry bag. The table can be carried like a suitcase by grasping the handle.

Warning: If the frame/legs/table structure is damaged in any way, the table should not be used. Using this table when damaged may cause harm or serious injury to you or your patient. It should be repaired immediately.



GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.